

# Packaging MDL Specimens for FedEx Pick-up.




Insert labeled vial into mailer



Insert the mailer into biohazard bag.



Remove FedEx receipt inside Lab Pack for your records.

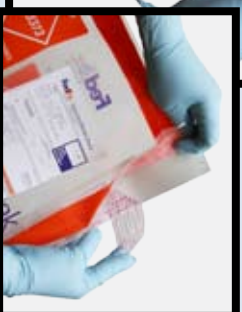

Don't forget to seal the Lab pack!

Insert sealed biohazard bag into the prepaid FedEx Lab Pack envelope. One Lab Pack will accommodate 6-7 Styrofoam containers.



Verify that a preaddressed airbill is attached to ensure proper delivery.

# Packaging MDL Specimens for FedEx Pick-up.

1

**Insert labeled vial into mailer**



**Insert the mailer into biohazard bag.**

2

3

**Insert sealed biohazard bag into the prepaid FedEx Lab Pack envelope. One Lab Pack will accommodate 6-7 Styrofoam containers.**



**Remove FedEx receipt inside Lab Pack for your records.**

**Don't forget to seal the Lab pack!**



**Verify that a preaddressed airbill is attached to ensure proper delivery.**

4