


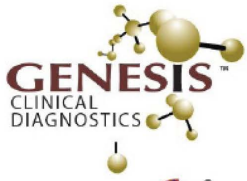

## Specimen Collection of Oropharyngeal Specimens:



- Collect oropharyngeal specimen **ONLY** using the oropharyngeal swab provided in the Novel Coronavirus Specimen Collection Kit.
- Immediately store collected oropharyngeal specimen in the refrigerator (4°C) until ready for transport. Specimen should be received within 72 hours of collection.

### Kit Contents Include:

- 1 **COVID-OneSwab™** viral transport media vial
- 1 Oropharyngeal swab
- 1 Biohazard bag
- 2 Cold Packs
- 1 Transport box

### Nasopharyngeal collection:

1. Insert swab into the posterior pharynx and tonsillar areas. Rub swab over both tonsillar pillars and posterior oropharynx and avoid touching the tongue, teeth, and gums.
2. Leave swab in place for several seconds to absorb secretions. Slowly remove swab while rotating it.
3. Insert swab into the **COVID-OneSwab™** viral transport media vial. Break the shaft at the molded break point.
4. Be sure the swab collected above has been inserted into the **COVID-OneSwab™** viral transport media vial and that the swab tip is immersed in the liquid. To prevent leakage, be sure the swab fits into the vial prior to capping. Tightly cap the vial and label with a minimum of two patient identifiers, including patient name and date of birth.
5. Place the labeled **COVID-OneSwab™** viral transport media vial in the central pocket of a biohazard bag and seal.
6. Insert a test requisition form in the front pocket of the biohazard bag.
7. Place inside of the transport box.
8. Place into the prepaid FedEx Lab Pack envelope. Be sure to seal the Lab Pack and verify that a preaddressed airbill is attached to ensure proper delivery. Contact FedEx at 1 (800) 463-3339 or visit [www.fedex.com](http://www.fedex.com) to arrange a pick-up for overnight delivery.