PROCEDURE FOR CLEAN CATCH URINES

Patient Instructions:

FEMALE

1. Remove all necessary garments.
2. Wash hands thoroughly with soap and water.
3. Separate labia (folds of skin covering opening from which you urinate) with one hand. With the other hand, take the provided cleansing towelette and cleanse the meatus (opening from which you urinate) and surrounding area from front to back. Discard towelette in wastebasket.
4. Remembering to keep the labia separated throughout the entire procedure, begin to urinate in the toilet, then catch a stream of urine directly in the collection cup provided. Avoid any contact with rim or inside of the cup.
5. Once an adequate amount of urine is collected, remove the cup from the urine stream and pass the remainder of your urine into the toilet bowl.



MALE

1. Remove all necessary garments.
2. Wash hands thoroughly with soap and water.
3. If uncircumcised, hold foreskin back with one hand. With the other hand use the provided cleansing towelette wipe the end of the penis well using a circular stroke from the center outward. Discard towelette in the wastebasket.
4. Remembering to keep the foreskin held back throughout entire procedure, begin to urinate in the toilet, then catch a stream of urine directly in the collection cup provided. Avoid any contact with rim or inside of the cup.
5. Once an adequate amount of urine is collected, remove the cup from the urine stream and pass the remainder of your urine into the toilet bowl.

Please give cup of urine to office personnel

Office Personnel Instructions:

1. Open the peel pouch of the **UroSwab**[®], remove the vial, and open the lid exposing the sponge swab.
2. Dip the sponge swab into the collection cup to allow the sponge to absorb urine.
3. Replace the sponge swab back into the **UroSwab**[®] transport vial.
4. To prevent leakage, tightly cap the vial and label with patient information.
5. For packaging and shipping instructions, please refer to MDL's catalog of services.

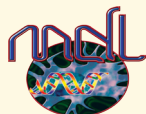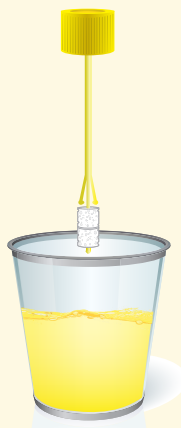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