

Packaging MDL ThinPrep Specimens for FedEx Pick-up.

1 Insert labeled vial into the tray.



2 Insert the tray into Ziplock bag.



3 Insert the sealed bag with tray into the mailer.



4 Be sure to include a completed test requisition form for each specimen in the top of the mailer.

5 Insert the closed mailer into the prepaid FedEx Lab Pack envelope.



6 Remove FedEx receipt inside Lab Pack for your records.

Don't forget to seal the Lab pack!



Verify that a preaddressed airbill is attached to ensure proper delivery.

Packaging MDL ThinPrep Specimens for FedEx Pick-up.

1 Insert labeled vial into the tray.



3 Insert the sealed bag with tray into the mailer.



2 Insert the tray into Ziplock bag.



4 Be sure to include a completed test requisition form for each specimen in the top of the mailer.



5 Insert the closed mailer into the prepaid FedEx Lab Pack envelope.



6 Remove FedEx receipt inside Lab Pack for your records.



7 Don't forget to seal the Lab pack!

8 Verify that a preaddressed airbill is attached to ensure proper delivery.

